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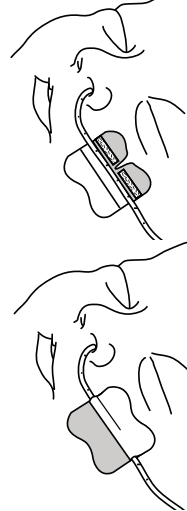
INSTRUCTIONS

EN English

- Select** the area for the placement of the Grip-Lok. Note: The feeding tube should be already inserted in the patient.
- Prepare** the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
- Remove** the bottom liner by holding fabric section and pulling-off liner. Do not remove the top liner at this time.
- Place** the Grip-Lok on the site by continuing to hold by the fabric and press onto the skin with gentle pressure. The top liner should now be removed.
- Position** the feeding tube in the center of the top exposed adhesive strip between the fabric and the white hook section.
- Secure** the feeding tube by folding over the fabric section until it meets with the white hook portion and apply gentle pressure to the back of the entire fabric area.

Note: to remove or adjust secured tube, hold hydrocolloid adhesive down to skin while opening top fabric section.

Note: the Grip-Lok 2100NGH may be cut in half prior to removal of liners to reduce the size.

Note: replace securement device if soiled or saturated in fluid.

Note: use of a sterile water swab will ease removal of securement device from the skin.

This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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