




## Catheter Securement

### Application of Grip-Lok Nasal Gastric Hydrocolloid Securement Device

Grip-Lok Nasal Gastric Hydrocolloid (2100NGH) offers firm securement of feeding tubes including pure silicone materials and will fit a wide range of tube sizes. Grip-Lok is applied in the same simple manner as a bandage and can be easily placed and adjusted with gloved hands. *Neonatal Skin Care Practice Guidelines* state the preference for hydrocolloid adhesives on neonatal patients.

- Soft and flexible fabric design improves patient comfort.
- Provides superior securement for both horizontal and vertical lifting accidental pulls.
- Simple to apply, inspect and adjust.
- Hypoallergenic and latex-free to reduce the risk of allergic reactions and skin irritation.

Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.

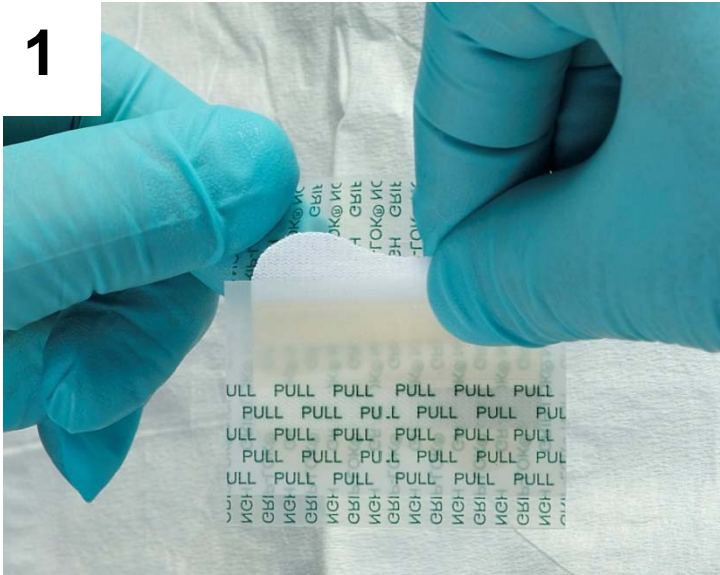

Remove the bottom liner by holding fabric section and pulling-off liner. It is not necessary to remove the top liner at this time.




Place the Grip-Lok on the site by continuing to hold by the fabric and press onto the skin with gentle pressure. The top liner should now be removed.



Position the feeding tube in the center of the top exposed adhesive strip between the fabric and the white hook section.



Secure the feeding tube by folding over the fabric section until it meets with the hook portion and apply gentle pressure to the back of the entire fabric area.

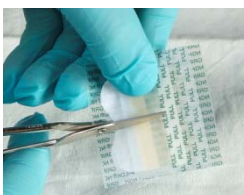

Note: the Grip-Lok 2100NGH may be cut in half prior to removal of liners to reduce the size.

