



Catheter Securement

Application of Grip-Lok for Feeding Tube Securement


Grip-Lok Small (3200S) is a line and catheter stabilization device that is strong enough to lock lines and tubes securely in place, yet versatile enough for almost any securement situation. Grip-Lok is applied in the same simple manner as a bandage and can be easily placed and adjusted with gloved hands.

- Soft and flexible fabric design improves patient comfort.
- Provides superior securement for both horizontal and vertical lifting accidental line pulls.
- Simple to apply, inspect and adjust.
- Hypoallergenic, breathable and latex-free to reduce the risk of allergic reactions and skin irritation.

Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.



Open the top flap and slide the Grip-Lok under the catheter, tube or line, centering it above the adhesive area.

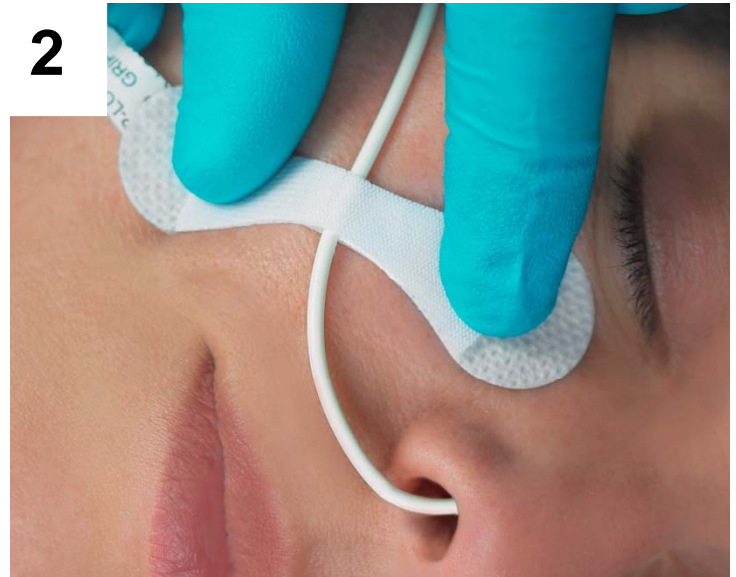

Pull the paper backing from one side of the Grip-Lok, then the other, to secure in desired position on the skin.



Remove the inside liner labeled PULL to expose the adhesive areas.



Place the tube on the bottom adhesive area and secure top flap over the tube. Gently press the top flap down around the edges of the tube to ensure the adhesive connects well to the tube and ensure the hook and loop have achieved the best possible connection.