




## Application of Grip-Lok WA (Wide Adhesive) for PICC Catheter Securement

Grip-Lok WA (3300M WA) is a catheter stabilization device that is strong enough to lock catheters securely in place, yet versatile enough for almost any securement situation. The adhesive on Grip-Lok WA will adhere and secure catheters made from silicone, PVC and other plastic compounds.

### Catheter Securement

- Soft and flexible fabric design improves patient comfort.
- Provides superior securement for both horizontal and vertical lifting accidental line pulls.
- Simple to apply, inspect and adjust.
- Hypoallergenic, breathable and latex-free to reduce the risk of allergic reactions and skin irritation.

Prepare the skin is according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.

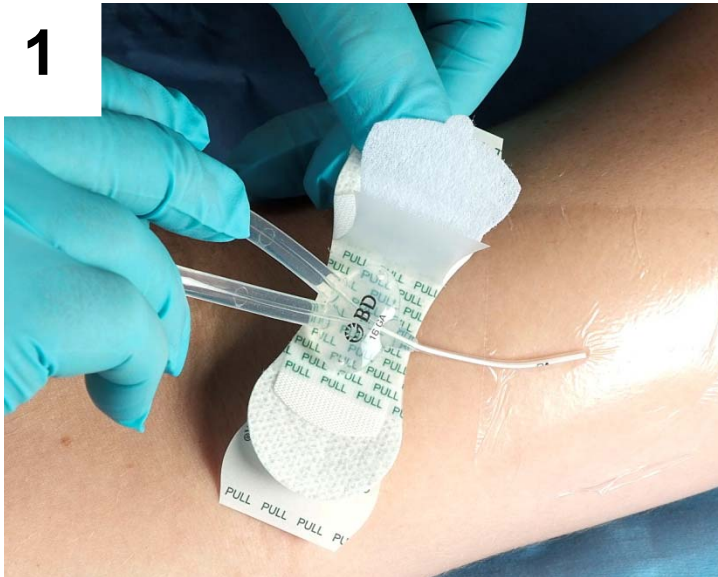

1 Open the top flap and slide the Grip-Lok under the catheter, tube or line, centering it above the adhesive area.



2 Pull the paper backing from one side of the Grip-Lok, then the other, to secure in desired position on the skin.



3 Remove the inside liner labeled PULL to expose the adhesive areas.



4 Place the catheter, tube or line on the bottom adhesive area and secure top flap over the catheter, tube or line.

**Grip-Lok WA will secure a wide variety of catheter hub sizes and shapes.**

