



## Catheter Securement

### Application of Grip-Lok for IV Line Securement

Grip-Lok Medium (3300M) is a line and catheter stabilization device that is strong enough to lock lines and tubes securely in place, yet versatile enough for almost any securement situation. Grip-Lok is applied in the same simple manner as a bandage and can be easily placed and adjusted with gloved hands.

- Soft and flexible fabric design improves patient comfort.
- Provides superior securement for both horizontal and vertical lifting accidental line pulls.
- Simple to apply, inspect and adjust.
- Hypoallergenic, breathable and latex-free to reduce the risk of allergic reactions and skin irritation.

Prepare the skin is according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.

1



Pull and remove one side of the bottom release liner and position the Grip-Lok under the IV Line. Then remove the other side of the bottom release liner.

2



Open the top Velcro® over-strap and remove inner release liner to expose adhesive.

3



Place the line/hub in the center of the bottom adhesive strip.

4



Secure the top strap over the line/hub and press the adhesive in place. The top strap can be peeled back to inspect and adjust the catheter line/hub.