



Catheter Securement


Application of Grip-Lok CS for the BD Arterial Cannula with FloSwitch®

The Grip-Lok 3301MCS-BD is an arterial cannula stabilization device that is strong enough to lock the cannula securely in place, yet versatile enough for almost any patient. The Grip-Lok CS provides a die-cut adhesive area that matches the shape of the arterial cannula hub for proper positioning and additional securement stability.

- Soft and flexible fabric design improves patient comfort.
- Provides superior securement for both horizontal and vertical lifting accidental line pulls.
- Simple to apply, inspect and adjust.
- Hypoallergenic, breathable and latex-free to reduce the risk of allergic reactions and skin irritation.

Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.

1

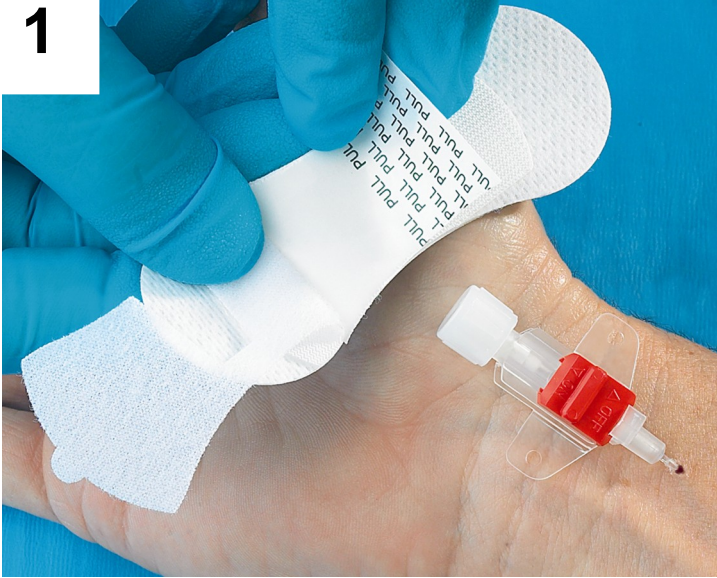

Open the top flap, remove the interior liner and hold the flap open.

2



Slide the Grip-Lok under the arterial cannula and adhere in the exposed adhesive cut-out space provided.

3



Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.

4



Secure the top flap over the arterial cannula with the FloSwitch portion coming through the center cut-out.

Directions: for added skin protection, skin prep may be used prior to application of securement device.

NOTE: replace securement device if soiled or saturated with fluid. Use of alcohol swab will ease removal of securement device from the skin.