# Universal Securement Device

REF	<b>Description</b>
33xxMCS Series	Medium Catheter Securement (MCS) for PICC/CVC Applications

Zefon International, Inc.  
5350 SW 1st Lane  
Ocala, FL 34474 USA

Phone: +1 352 854 8080  
Fax: +1 352 854 7480  
www.zefon.com

EC REP

Blue Box Medical Limited  
Unit 29, New Forest Enterprise Centre,  
Rushington Business Park  
Chapel Lane  
Totton, Southampton, SO40 9LA U.K.

Phone: +023 8066 9000  
Fax: +023 8066 9002  
www.blueboxmedical.co.uk

STERILE R

LV05015 Rev.2




Grip-Lok is a registered trademark of Zefon International.

## INSTRUCTIONS

EN

English

1. **Select** the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
2. **Prepare** the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
3. **Open** the top flap, remove the interior liner and hold the flap open.
4. **Slide** the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
5. **Secure** the top flap over the catheter.
6. **Hold** the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.

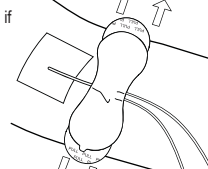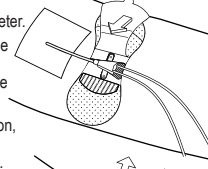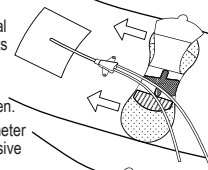**Direction:** for added skin protection, skin prep may be used prior to application of securement device.

**Note:** replace securement device if soiled or saturated in fluid.

**Note:** use of an alcohol swab will ease removal of securement device from the skin.

This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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