



Catheter Securement

Application of Grip-Lok CS for Arrow CVC Securement

Grip-Lok 3303MCS-TA is a catheter stabilization device that is strong enough to lock catheters securely in place, yet versatile enough for almost every patient. The Grip-Lok CS provides a foam die-cut insert that matches the shape of the catheter hub for proper positioning and additional securement stability.

- Soft and flexible fabric design improves patient comfort.
- Provides superior securement for both horizontal and vertical lifting accidental line pulls.
- Simple to apply, inspect and adjust.
- Hypoallergenic, breathable and latex-free to reduce the risk of allergic reactions and skin irritation.


Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.



Open the top flap and slide the Grip-Lok under catheter hub. Position the catheter hub down into the foam cut-out and press gently to the adhesive.



Pull and remove one side of the bottom release liner while holding Grip-Lok in position. Then pull and remove the other side of the bottom release liner.

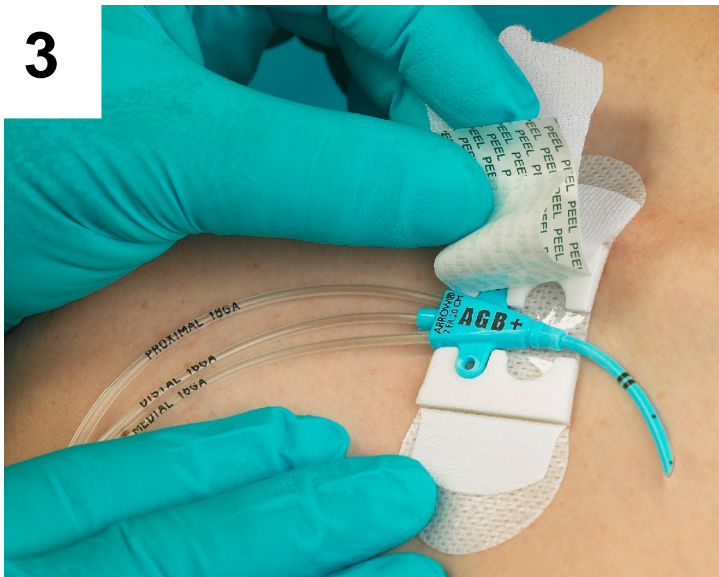

Pull and remove the inner release liner under the top flap to expose adhesive.



Secure the top strap over the catheter hub and press the adhesive in place. The top strap can be peeled back to inspect and adjust the catheter hub.



Grip-Lok CS for Arrow CVC securement will also fit alternative catheter attachment points.