Foley Catheter Securement Device

REF

Description

3400LFC Foley Catheter Securement Device

STERILE R

LV05010 Rev 4

Zefon International, Inc.
5350 SW 1st Lane
Ocala, FL 34474 USA

Phone: +1 352 854 8080
Fax: +1 352 854 7480
www.zefon.com

EC REP

Blue Box Medical Limited
Unit 29, New Forest Enterprise Centre,
Rushington Business Park
Chapel Lane
Totton, Southampton, SO40 9LA, U.K.


Phone: +023 8066 9000
Fax: +023 8066 9002
www.blueboxmedical.co.uk

Grip-Lok is a registered trademark of Zefon International.

INSTRUCTIONS

EN
English

1. **Select** the area for the placement of the Grip-Lok. Note: The catheter should be already inserted.
2. **Prepare** the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
3. **Hold** the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
4. **Open** and separate the top fabric straps.
5. **Place** the drainage lumen between the foam strip and the back of the fabric straps.
6. **Position** the catheter so that the Y-point is centered at the keyhole in the top fabric straps.
7. **Secure** one of the fabric straps over the drainage lumen and the other over the catheter below the Y-point.
8. **Adjust** the securement of the catheter by applying the strap attachments tighter or looser as needed.

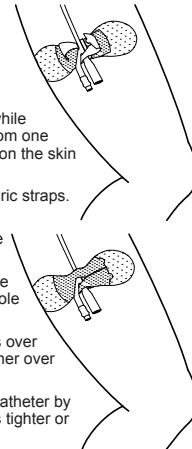Direction: for added skin protection, use skin prep pad prior to application of stabilization device.
Note: replace securement device if soiled or saturated in fluid.
Note: use of an alcohol swab will ease removal of securement device from the skin.

This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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