
4200ZAB Arterial Catheter Securement

REF	Description
4200ZAB	Arterial Catheter Securement

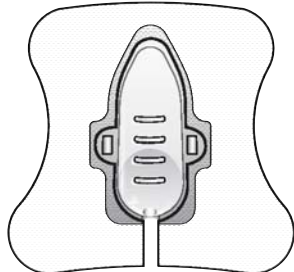STERILE R

LV07802 Rev 3

Zefon International, Inc.
5350 SW 1st Lane
Ocala, FL 34474 USA

Phone: +1 352 854 8080
Fax: +1 352 854 7480
www.zefon.com

Blue Box Medical Limited
Unit 29, New Forest Enterprise Centre,
Rushington Business Park
Chapel Lane
Totton, Southampton, SO40 9LA, U.K.




Phone: +023 8066 9000
Fax: +023 8066 9002
www.blueboxmedical.co.uk

Grip-Lok is a registered trademark of Zefon International.

INSTRUCTIONS EN

English

1. **Prepare** the skin around the arterial catheter according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
2. **Remove** center layer of bandage backing to expose adhesive.
3. **Engage** the pressure line in the stabilizing collar by sliding the line up the channel to the notch.
4. **While** holding the transparent dome, apply even pressure over the adhesive surface. Remove the bandage backing on the outer edges and smooth around wrist.
5. **Secure** the pressure line along the guide hook on the side of the dome.

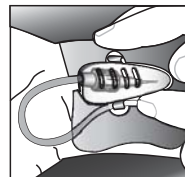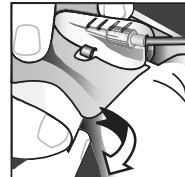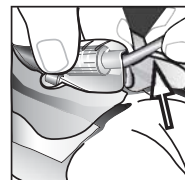Changing the securement device: Remove by reversing steps 3-5 taking care not to dislodge the catheter. **Note:** For added skin protection, use skin prep pad prior to application of securement device. **Note:** Replace securement device if soiled or saturated in fluid. **Note:** Use of an alcohol swab will ease removal of securement device from the skin.

This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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