Securement Device for PICC/CVC

REF

Description

33xxMCS Series Medium Catheter Securement (MCS) for PICC/CVC Applications

Grip-Lok is a trademark of Zefon International, Inc.

STERILE R

LV05015 Rev.1

Zefon International, Inc.
5350 SW 1st Lane
Ocala, FL 34474 USA

Phone: +1 352 854 8080
Fax: +1 352 854 7480
www.zefon.com

EC REP

Blue Box Medical Limited
Unit 29, New Forest Enterprise Centre,
Rushington Business Park
Chapel Lane
Totter, Southampton, SO40 9LA

Phone: +023 8066 9000
Fax: +023 8066 9002
www.blueboxmedical.co.uk

INSTRUCTIONS




English

- Select** the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
- Prepare** the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
- Open** the top flap, remove the interior liner and hold the flap open.
- Slide** the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
- Secure** the top flap over the catheter.
- Hold** the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.

Direction: for added skin protection, skin prep may be used prior to application of securement device.

Note: replace securement device if soiled or saturated in fluid.

Note: use of an alcohol swab will ease removal of securement device from the skin.

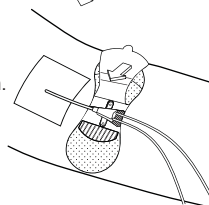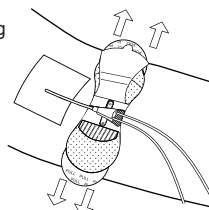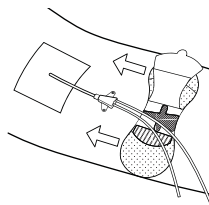This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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