## Universal Securement Device

REF	Description
3100ESH	Extra Small Hydrocolloid Securement
3200SH	Small Hydrocolloid Securement

STERILE R

LV07904 Rev 1

Zefon International, Inc.  
5350 SW 1st Lane  
Ocala, FL 34474 USA

Phone: +1 352 854 8080  
Fax: +1 352 854 7480  
www.zefon.com

EC REP

Blue Box Medical Limited  
Unit 29, New Forest Enterprise Centre,  
Rushington Business Park  
Chapel Lane  
Totton, Southampton, SO40 9LA

Phone: +023 8066 9000  
Fax: +023 8066 9002  
www.blueboxmedical.co.uk

Grip-Lok is a registered trademark of Zefon International.

## INSTRUCTIONS


English

1. Select the correct size Grip-Lok for your application.

REF	Size	Line/Tubing Diameter
3100ESH	Extra Small	1 to 3 mm
3200SH	Small	1.5 to 4.5 mm

2. Select the area for the placement of the Grip-Lok. Note: The line or tube should be already inserted in the patient.
3. Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
4. Remove the liner from the bottom of the Grip-Lok by lifting from the center.
5. Place the Grip-Lok back onto the liner with one half extending over the edge. This provides an easy way to hold and apply to the skin.
6. Open the top flap, remove the interior liner and hold the flap open.
7. Slide the Grip-Lok under so that the line or tube is in the center of the exposed adhesive.
8. Secure the top flap over the line or tube.
9. Hold the Grip-Lok in position on the skin while removing the bottom liner. Apply gentle pressure to the center fabric area over the adhesive.

Note: replace securement device if soiled or saturated in fluid.  
Note: use of a sterile water swab will ease removal of securement device from the skin.

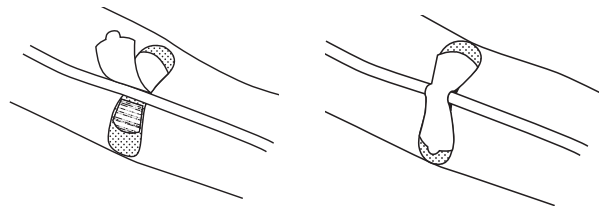This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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