Universal Securement Device

REF

Description

3300MWA Medium Universal Securement with Wide Adhesive area

STERILE R

LV05009 Rev 4

Zefon International, Inc.
5350 SW 1st Lane
Ocala, FL 34474 USA

Phone: +1 352 854 8080
Fax: +1 352 854 7480
www.zefon.com

EC REP

Blue Box Medical Limited
Unit 29, New Forest Enterprise Centre,
Rushington Business Park
Chapel Lane
Totton, Southampton, SO40 9LA


Phone: +023 8066 9000
Fax: +023 8066 9002
www.blueboxmedical.co.uk

Grip-Lok is a registered trademark of Zefon International.

INSTRUCTIONS

English

- Select** the area for the placement of the Grip-Lok. Note: catheter, tube or line should be already placed/inserted.
 - Prepare** the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
 - Open** the top flap and slide the Grip-Lok under the catheter, tube or line, centering it above the adhesive area.
 - Pull** the paper backing from one side of the Grip-Lok, then the other, to secure in desired position on the skin.
 - Remove** the interior liner to expose the adhesive areas.
 - Place** the catheter, tube or line in the center of the exposed adhesive.
 - Secure** top flap over the catheter, tube or line and apply gentle pressure to the center fabric area over the adhesive.
- Direction:** for added skin protection, use skin prep pad prior to application of stabilization device.
- Note:** replace securement device if soiled or saturated in fluid.
- Note:** use of an alcohol swab will ease removal of securement device from the skin.

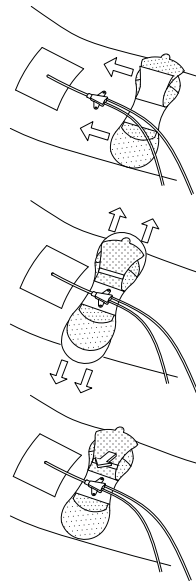This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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