InfuseIT™

Pressure Infusor

	500ml	1000ml	3000ml
REF	ZIT-500	ZIT-1000	ZIT-3000
	ZIT-502	ZIT-1002	ZIT-3002

Zefon International, Inc.
5350 SW 1st Lane
Ocala, FL 34474 USA

Phone: +1 352 854 8080
Fax: +1 352 854 7480
www.zefon.com

EC REP

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INSTRUCTIONS

English

- SELECT the correct size infusor for the IV fluid bag.
 - INSERT IV fluid bag between the pressure infusor and sleeve.
 - HANG IV fluid bag on the infusors plastic hook.
 - HANG pressure infusor on IV pole or other location IV fluid bags are normally hung.
 - Inflation using Thumbwheel Model
 - Turn the thumbwheel clockwise until snug. This is the fully CLOSED position. DO NOT over tighten.
 - SQUEEZE inflation bulb repeatedly until a pressure of 300mm Hg is achieved. DO NOT over pressurize.
 - Inflation using Stopcock Model
 - TURN the stopcock valve so the arrow points to the right. This is the INFLATE position.
 - SQUEEZE the inflation bulb repeatedly until a pressure of 300mm Hg is achieved. DO NOT over pressurize.
 - TURN the stopcock valve so the arrow points UP (towards the infusor). This is the HOLD position. The stopcock valve is required to be in this position for as long as pressure is to be held.
 - Deflation using Thumbwheel Model
Turn the thumbwheel counter-clockwise to the fully OPEN position
 - Deflation using Stopcock Model
Turn the stopcock valve so the arrow points down (towards the inflation bulb). This is the DEFLATE position.
- Cleaning Instructions :**
Note: Do not soak or immerse the infusor. InfuseIT manual pressure infusors may be cleaned by using one of the following methods:
APPLY either a 10% bleach solution or isopropyl alcohol to the infusor.
USE a soft brush or cleaning cloth to remove stains.
RINSE thoroughly with running water. Ensure the vent hole in the gauge is down so water doesn't enter the gauge.
ALLOW infusor to air dry before use.
Alternately, the following products can be safely used to clean the infusor when following the manufacturer's directions:
Sporicidin®, PDI® Super Sani-Cloth® wipes

Federal law (USA) restricts this device to sale by or on order of a physician.

This product does not contain any natural rubber latex.

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Sporicidin®, Salviette PDI® Super Sani-Cloth®

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Sporicidin®, Toallitas PDI® Super Sani-Cloth®

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Grip-Lok is a registered trademark of Zefon International.