Reusable Blood Pressure Cuffs Cleaning, Disinfecting and Usage Instructions

DuraCheck™ Reusable Blood Pressure Cuffs

Size	1 Tube	2 Tube	Manual
Thigh	ZDC4001	ZDC4002	ZDC4012
Large Adult Long	ZDC4121	ZDC4122	ZDC4132
Large Adult	ZDC4101	ZDC4102	ZDC4112
Adult Long	ZDC4221	ZDC4222	ZDC4232
Adult	ZDC4201	ZDC4202	ZDC4212
Small Adult	ZDC4301	ZDC4302	ZDC4312
Child	ZDC4401	ZDC4402	ZDC4412
Infant	ZDC4901	ZDC4902	ZDC4912

VitalCheck™ Reusable Blood Pressure Cuffs

Size	1 Tube	2 Tube	Manual
Thigh (brown)	ZVC4001	ZVC4002	ZVC4012
Large Adult (plum)	ZVCC4101	ZVCC4102	ZVCC4112
Large Adult Long(plum)	ZVCC4121	ZVCC4122	ZVCC4132
Adult (dark blue)	ZVCC4201	ZVCC4202	ZVCC4212
Adult Long (dark blue)	ZVCC4221	ZVCC4222	ZVCC4232
Small Adult (light blue)	ZVCC4301	ZVCC4302	ZVCC4312
Child (green)	ZVCC4401	ZVCC4402	ZVCC4412
Infant (brown)	ZVC4901	ZVC4902	ZVC4912

DuraCheck and VitalCheck are trademarks of Zefon International.

LV05001 Rev 12

Zefon International, Inc.
5350 SW 1st Lane
Ocala, FL 34474 USA

Phone: +1 352 854 8080
Fax: +1 352 854 7480
www.zefon.com

Blue Box Medical Limited
Unit 29, New Forest Enterprise Centre,
Rushington Business Park
Chapel Lane
Totton, Southampton, SO40 9LA U.K.

Phone: +023 8066 9000
Fax: +023 8066 9002
www.blueboxmedical.co.uk

Measuring Blood Pressure

1. **READ** blood pressure monitor instructions prior to use.
2. **SELECT** the correct size blood pressure cuff by measuring patient's limb and comparing to the stated size range on the cuff. If limb circumference falls between 2 cuff sizes, choose the larger size cuff.
3. **SELECT** the limb of the patient to be used for the blood pressure measurement. Normally the upper arm between the patients elbow and shoulder is generally preferred.
4. **APPLY** the cuff to the selected limb area of the patient with the artery marking arrow over the brachial or femoral artery.
5. Ensure that the **INDEX** line falls within the specified **RANGE** area on the inner side of the cuff. If it does not, choose a larger or smaller cuff that fits correctly.
6. **SECURE / FASTEN** cuff snugly. A properly secured cuff will have a snug fit, but still allow a finger to fit between the cuff and patient.

Cleaning & Disinfecting

Note: For Vitalcheck brand cuffs, remove the inner bladder from the cuff sleeve during cleaning.

1. **PREPARE** a cleaning detergent such as Enzol, Cidezyme or Burnishine and a disinfecting solution such as Cidex, Isopropyl Alcohol, Sporicidin or 10% bleach solution in separate containers. Follow manufacturer instructions for proper solution preparation.
2. **APPLY** liberal amounts of the cleaning detergent on the cuff and allow it to sit for a minimum of 1 minute.
Note: If cuff is to be submerged or soaked in liquid, cuff tubes should be capped. Liquid must not be introduced into the cuff tubes.
3. **USE** a soft brush or cleaning cloth to remove stains.
4. **RINSE** thoroughly with water.
5. **APPLY** disinfecting solution on the cuff until fully saturated. Let sit for a minimum of 5 minutes.
6. **RINSE** thoroughly with water.
7. Allow cuff to **AIR DRY** before use.

Caution: Failure to adequately rinse the cuff with water after cleaning or disinfection may result in residual cleaning or disinfecting solution irritating the patients skin.

Warning: Never allow any liquid to enter the tubes of the cuffs. If liquid has entered the cuff tubes, it should be discarded.

Warning: If the cuff has been grossly contaminated with blood or other bodily fluids, it should be discarded.

Federal law (USA) restricts this device to sale by or on order of a physician.

This product does not contain any natural rubber latex.

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